

Clinical Work Flows

Appt.	Immediate Denture	Reference Denture	Pt. Has No Denture
1	Impressions or models, and bite registration to desired VDO	Use pt's existing denture as a custom tray. Make final wash impression in the denture; bite reg. to desired VDO	Make preliminary impressions. We return occlusal rim(s)
2	Final denture insertion	Functional try-in* returned for evaluation	In occlusal rim – make final wash impression in rim as well as bite to VDO
3		Final denture insertion	Functional try-in* returned for evaluation
4			Final denture insertion

*Try in is printed in tooth shade acrylic and may be worn by the patient at home to determine how well it functions. As the teeth are not in wax, any esthetic or bite adjustments can be made by grinding, marking, or adding wax to relate desired changes. Minor fit adjustments may be made by grinding but for major adjustments make a light bodied wash impression in the try in. All adjustments will be incorporated in the final denture.

6 Printed Tooth Shades: A1 A2 A3 B1 B2 BL3 (standard denture)

16 IPN 3D™ Digital Denture Teeth Shades + BL1 & BL3 (premium denture)



5 Lucitone Base Shades

